### Garantiezusage

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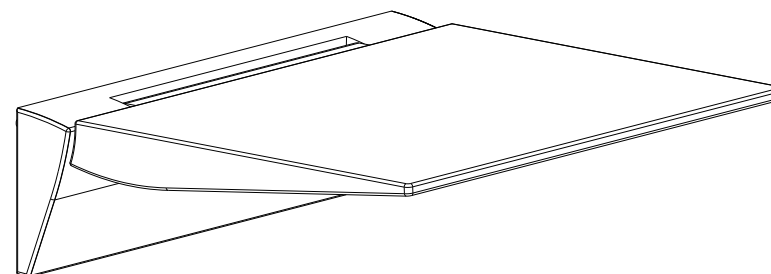
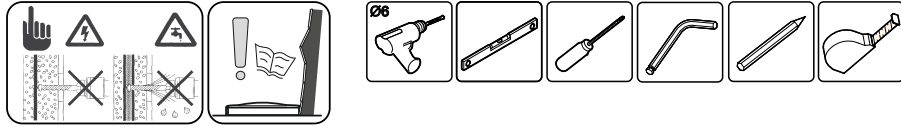
#### **GB** Cleaning and maintenance

1. Do not use scouring or aggressive detergents, or petrol or thinner.
2. We recommend a general household detergent or "Mister Proper". After cleaning rinse well with water.
3. We recommend for lime deposit a vinegar essence (25%) or household vinegar.
4. Do not use shower gels that are aggressive to polystyrol synthetic materials.

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







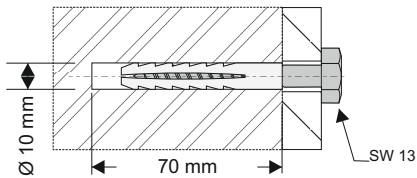
- GB** **STANDART FIXING:**  
The following material is enclosed for standard fixing: Nylon plug 10 mm and screw 8 x 60 mm suitable for: reinforced concrete R250. For walls not listed above a specialist in fixing technology must be contacted.

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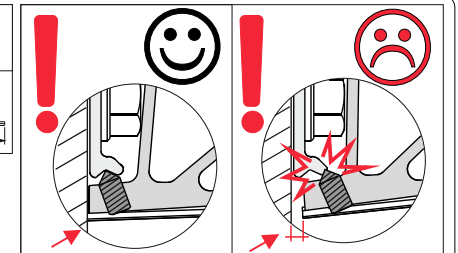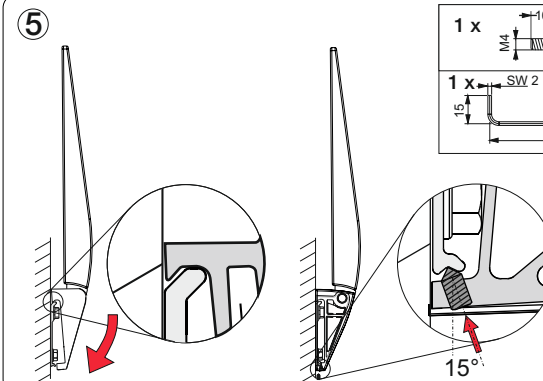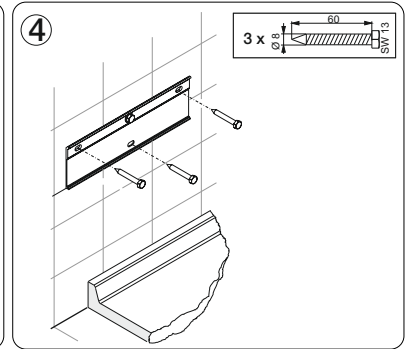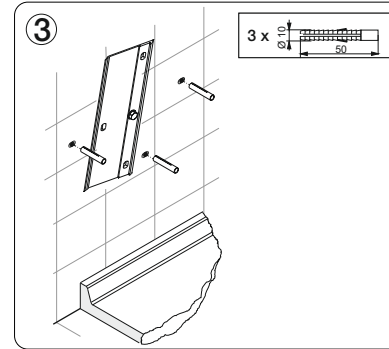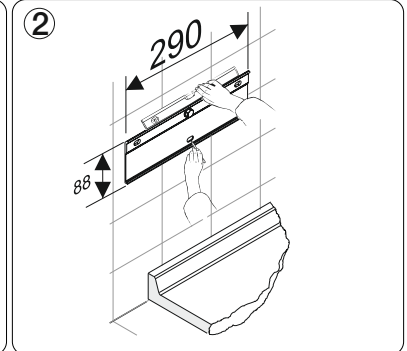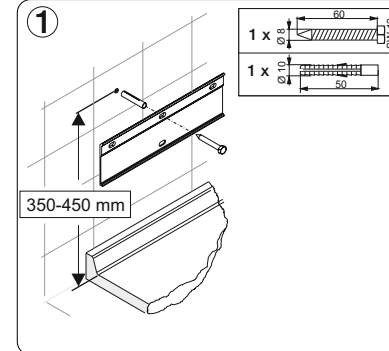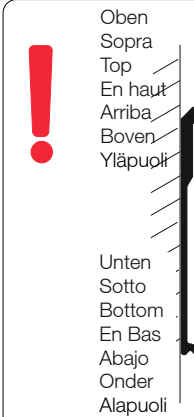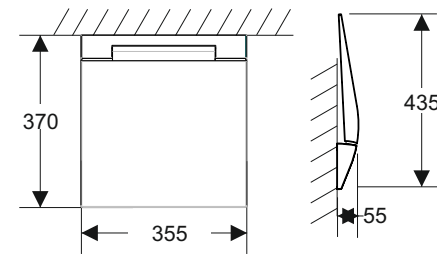- GB** **IMPORTANT POINTS:**  
The shower seat has been designed to be fitted inside a shower cubicle or over a bath. IT MUST BE FITTED TO A SOLID WALL OR OTHER FIRM SURFACE. The wall face must be flat. The installer should ensure that the nature and construction of the wall will provide a firm fixing for all 4 screws supporting the shower seat.  
**FIXING HEIGHT:**  
The fixing height of the shower seat depends largely on the height of the user, it will usually be somewhere between 450 - 550 mm above the bottom of the shower tray or bath. A simple method of determining the shower seat height would be to measure the seat height of the users favourite / most comfortable chair, and fix the shower seat at a similar height.

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