Serie 300

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

Cleaning and maintenance

1. Do not use scouring or aggressive detergents, or petrol or thinner.
2. Koch recommends a general household detergent or „Mister Propper“. After cleaning rinse well with water (max. 60°).
3. Koch recommends for lime deposit a vinegar essence (25%) or household vinegar.
4. Do not use shower gels as there are aggressive to polystyrol synthetic materials.

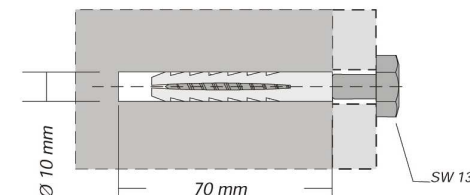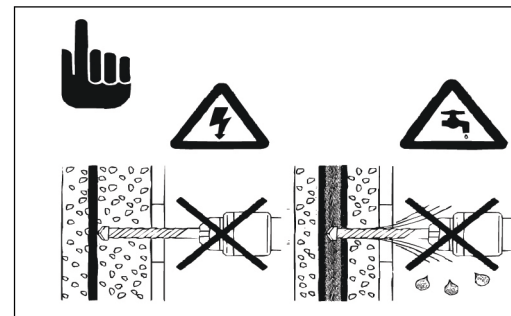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STANDART FIXING:

The following material is enclosed for standard fixing: Nylon Plug 10 mm, screw 8x60 mm. Suitable for: Reinforced Concrete R250.
For walls not listed above a Specialist in Fixing technology must be contacted.

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









IMPORTANT POINTS:

The shower seat has been designed to be fitted inside a shower cubicle or over a bath. It must be fitted to a solid wall or other firm surface: The wall face must be flat. The installer should ensure that the nature and construction of the wall will provide a firm fixing for all 4 wall fixing screws supporting the shower seat.

FIXING HEIGHT:

The fixing height of the shower seat depends largely on the height of the user, it will usually be somewhere between 450-550 mm above the bottom of the shower tray or bath.

(A simple method of determining the shower seat height would be to measure the seat height of the users favourite / most comfortable chair and fix the shower seat at a similar height.)

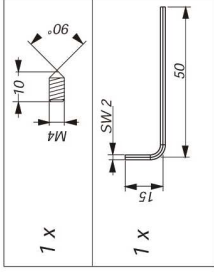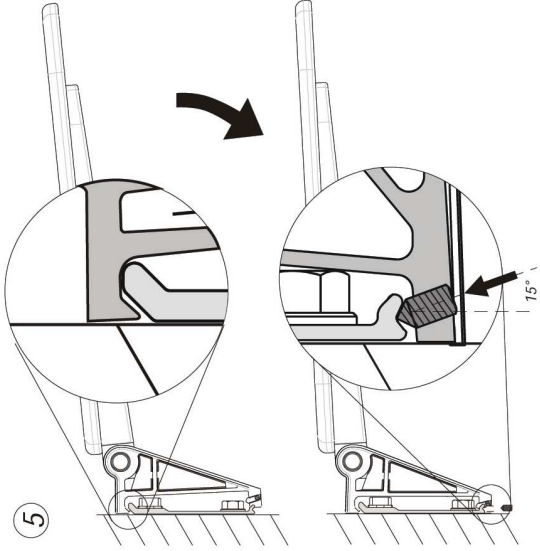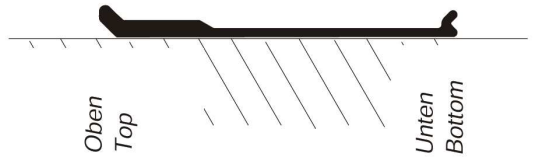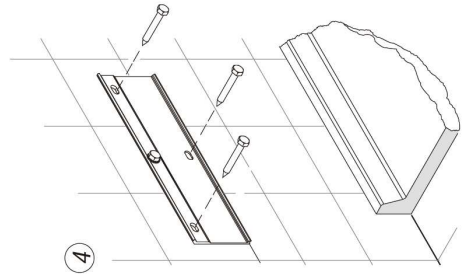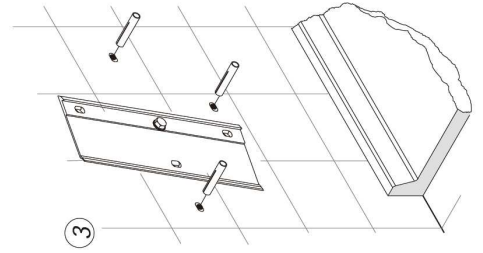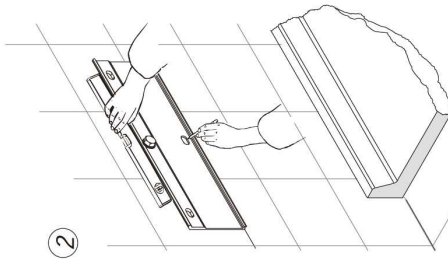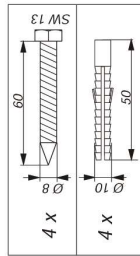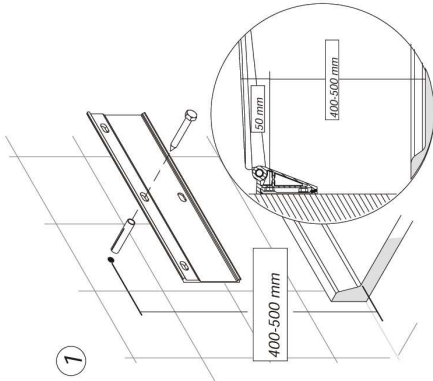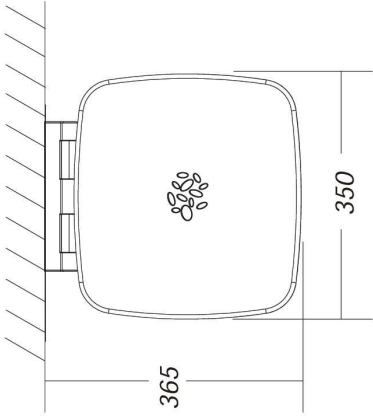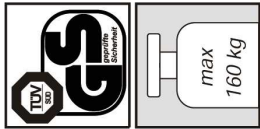Pos	No. articolo	quant.	Material
1	0030BZ00F	2	INOX
2	0080BF00F	4	INOX A2-70
3	0081BF08F	4	PA6
4	0957AL00G	1	AMG507 F27
5	1200KS**F	1	...
6	0098BF00F	1	INOX
7	1206KS**FL	1	ABS
8	1206KS**FR	1	ABS
9	1820AL**G	1	AMG505 F22